



BIORAD®

RELIABLE



Endoscope And Medical
Instrument Disinfection



FORMS OF PACKAGE

Description	Package
5 litre PE drum	4 x 5 litre
10 litre PE drum	1 piece

CHEMICAL COMPOSITION:

Bis (3-aminopropyl)dodecylamine, Didecylmethylpoly(oxethyl) Ammonium Propionate, corrosion inhibitor and auxiliary agents

PRODUCT SPECIFICATIONS:

RELIABLE is a ready-to-use and high active disinfectant for equipment. It is colourless, odour-free and nontoxic disinfectant containing no phenol, chlorine and formaldehyde and it is used for disinfection of medical equipment, flexible and rigid endoscopes. As it contains corrosion inhibitor, it is conveniently used on surfaces with metal accessories. It disinfects and sterilizes in 5 minutes and is usable for maximum 14 days without loss of activity. It can be easily checked with test strips.

INTENDED USE:

Reliable is very suitable for disinfection for all kinds of surgical equipment, heat-sensitive anaesthetic accessories (ambu mask, etc.) and all flexible and rigid endoscopes such as Karl Storz, Olympus, Pentax, Fujinon, as well as dental surgical materials, drills and laparoscopy equipment. It does not cause any harm on the endoscope covers such as rust, corrosion, softening, colour change, etc. Reliable solution is suitable for disinfection of medical equipment made of rubber, plastic, PVC, acrylic, polymer, polycarbonate, polystyrene, HDPE, porcelain and stainless steel

USAGE:

It is suitable for use with endoscope washing machines and for manual use. Fill the disinfection tank with solution. Disassemble the equipment precleared with enzymatic solution and place in the tank in such a way that all surfaces will contact with solution, and seal the tank. After waiting for its activation in the sealed tank, take out the equipment from the tank and rinse with ample water. You should check activity of the solution with test strips. When solution gets dirty noticeably, you are recommended to replace it. Please comply with the userinstructions of the manufacturer when use it with automatic endoscopy washing machine and disinfection machine.

ANTI-MICROBIAL CHARACTERISTICS:

Bactericidal, Fungicidal, Sporocidal, Virucidal, Tuberculocidal, Micobactericidal